



# ReVeneer™

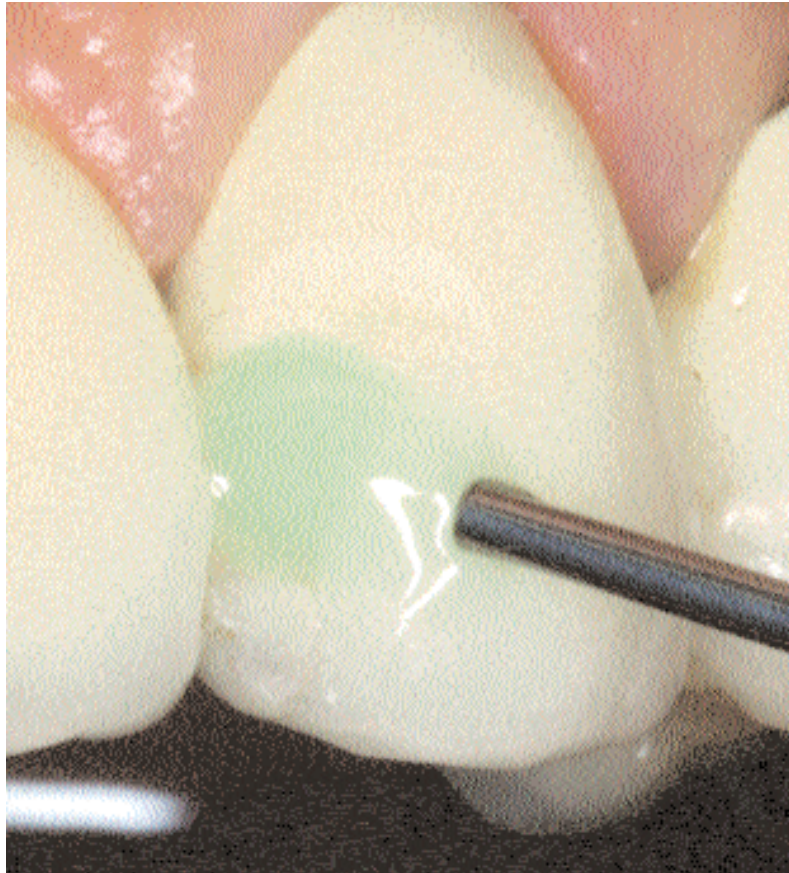


- rescue for  
ceramic fractures

# No panic,

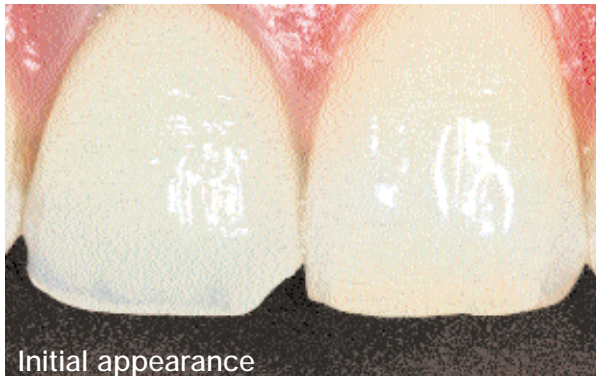

now there is **ReVeneer™** and the problem loses its terror.  
Relief for dentist and patient. **ReVeneer™** is the completely new and ingenious system

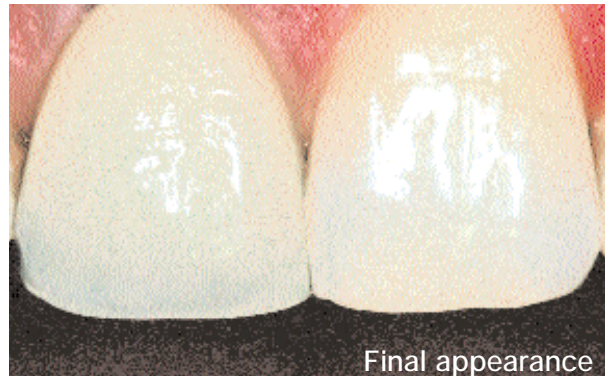

consisting of primer, repair paste and composite. In most cases, it saves you from having to refabricate a crown or bridge.

**Aesthetic, lasting, safe and acid-free.**

## Advantages of ReVeneer™:

- no acid etching (neither hydro fluoric nor phosphoric acid)
- safe intraoral use therefore possible
- no silane as primer (hydrolytic degradation of this primer in the mouth is possible)
- extremely high bonding
- time saving
- can be used universally
- tested and optimised on all currently used ceramics (feldspath and glass ceramic, press ceramic, zirconia, aluminium oxide)
- use of conventional curing lamps (halogen, LED, plasma light) possible

**Two unique product advantages lead securely to an aesthetically impeccable result.**

Because of ReVeneer Base's green colour, exact point application is possible. The change in colour from green to clear shows when curing is complete.



Dr. Gerrit Meyer MD of the University Clinic is enthusiastic: „**The green colour of the paste proved to be very positive as it provides a nice contrast to the ceramic and thus makes it easier to work precisely** “.



Ceramic repair in practice with **ReVeneer™**  
**Clinical case of a ceramic fracture**



Initial appearance



Roughening the surface with diamonds




Applying the **ReVeneer™** primer



After aeration of 2 minutes  
Applying the **ReVeneer™** paste



Painting on the repair paste

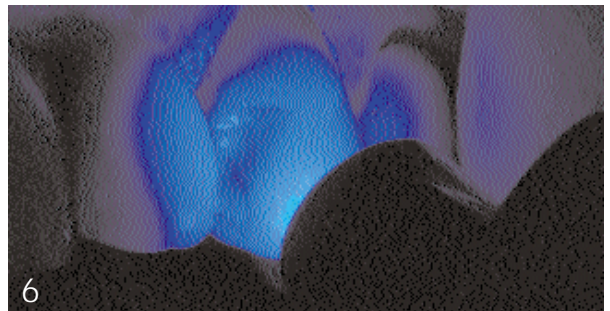

Curing 40 sec



After composite layering (shade A2) the surface  
is polished



Final appearance

## Packaging units ReVeneer™ system

### Sets

Adhesive set (Art. no.: 232205)

- 1 syringe (1g) ReVeneer™ base
- 1 bottle (3ml) ReVeneer™ primer
- 12 application tips (0.9x13mm)
- 1 brush holder
- 6 disposable brushes



ReVeneer repair set (Art. no.: 232220)

- 1 syringe (1g) ReVeneer™ base
- 1 bottle (3ml) ReVeneer™ primer
- 12 application tips (0.9x13mm)
- 1 dental brush holder
- 6 disposable brushes
- 1 syringe (3g) opaquer universal light
- 1 syringe (3g) opaquer universal dark
- 6 Fillis composite A3.5 with 0.3g
- 3 Fillis composite A3 with 0.3g
- 2 Fillis composite A2 with 0.3g
- 1 Filli composite B2 with 0.3g



### Refills

Art. no.:	Description of article	Containing
232210	ReVeneer™ base	1.0 g
232211	ReVeneer™ primer	3.0 ml
232220	ReVeneer™ opaquer universal light	3.0 g
232221	ReVeneer™ opaquer universal dark	3.0 g
232233	ReVeneer™ composite A3.5	3 Fillis with 0.3 g
232232	ReVeneer™ composite A3	3 Fillis with 0.3 g
232231	ReVeneer™ composite A2	3 Fillis with 0.3 g
232236	ReVeneer™ composite B2	3 Fillis with 0.3 g

