

# WELDER-spot welding system Instruction manual

---



## **Schütz-Dental GmbH**

Dieselstr. 5-6

61191 Rosbach/Germany

**Telefon: +49 (0) 6003 814-0**

**Telefax: +49 (0) 6003 814-900**

## Contents

Chapter	Page
<b>1 Safety.....</b>	<b>3</b>
1.1 Safety instructions.....	3
1.2 Safety measures (undertaken by the operator).....	3
1.3 Safety regulations.....	3
1.4 Built-in safety systems.....	4
<b>2 General danger information.....</b>	<b>4</b>
2.1 Danger areas .....	4
2.2 Operating and maintenance personnel.....	4
2.3 Installation of spare and wearing parts.....	5
2.4 Switching the device off.....	5
<b>3 Short description.....</b>	<b>6</b>
<b>4 Technische Daten des Gerätes.....</b>	<b>6</b>
<b>5 Transport and packaging.....</b>	<b>7</b>
<b>6 Set up/Starting the operation.....</b>	<b>8</b>
6.1 Starting the operation.....	8
<b>7 Operating elements.....</b>	<b>9</b>
7.1 Operational field.....	9
<b>8 Operation.....</b>	<b>10</b>
8.1 Selecting the programm.....	10
8.2 Changing a programm.....	10
8.3 Save programmes.....	11
8.4 Microscope.....	11
8.5 Sequence of operations.....	11
<b>9 Maintenance / Cleaning.....</b>	<b>13</b>
9.1 Cleaning .....	13
<b>10 Emergency.....</b>	<b>13</b>
<b>11 Demounting / disposal.....</b>	<b>13</b>
<b>12 Spare parts.....</b>	<b>13</b>
<b>13 Guarantee.....</b>	<b>14</b>

# 1 Safety

## 1.1 Safety instructions

The accident prevention regulations (VBG) of the responsible professional associations must be observed!

**Warning!**

It is strictly prohibited to make safety devices inoperable!

## 1.2 Safety measures (undertaken by the operator)

These operating instructions form an integral part of the machine.

- They must be available to the operating personnel at all times.
- They must be read and understood before the machine is set into operation.
- The corresponding references to danger and information must be observed.

Requirements of the owner regarding the machine operators and maintenance personnel:

- They must be instructed with regard to the machine's protective devices.
- They must be trained with regard to safe working methods.
- The observation of safety measures must be monitored.

Personnel must be provided with the appropriate tools and testing facilities.

## 1.3 Safety regulations

The system was built in compliance with German regulations.

**Harmonised standards:**

EN 50199:1995

EN 61000-3-2:2000

EN 61000-3-3:1995+A1:2001

## 1.4 Built-in safety systems

- a main switch on the front side of the system. It separates the system from the power supply

## 2 General danger information

### 2.1 Danger areas

A distance of at least 0.5 m from the machine is defined as a danger area as this is the space that it required by operating personnel to complete their work.

**Note !**

The protection of the machine becomes unreliable if it is not operated in keeping with these operating instructions.

The objekt to be worked on can be positioned freely inside the welding chamber. The position of the objekt can be optimized manually, monitored by microscope. The user's eyes are protected from flashes by a special shutter inside the microscope.

### 2.2 Operating and maintenance personnel

Operating personnel are people who are responsible for the transport, installation, operation and cleaning of the machine and for remedying faults.

1. The machine must only be operated by competently trained and authorised personnel.
2. The responsibilities associated with the operation of the machine must be clearly defined and observed to ensure that there are no uncertainties regarding competence in connection with the safety aspects.
3. The switching-off procedure defined in these operating instructions must be observed in connection with all work (operation, cleaning, exchanging spare and accessory parts).
4. The work to clean the machine and exchange spares and accessories has been described in these operating instructions in such a manner that they can be understood by people with specialist training in:
  - Electrical engineering / electronics
  - Mechanics / cleaning

5. Operators must refrain from all work that could impair the safety of the machine.
6. The operator must ensure that no unauthorised people work with the machine.
7. The operator must immediately notify the owner of any changes to the machine that could impair its safety.
8. The owner is obliged to always maintain a perfect state of the device.

### **2.3 Installation of spare and wearing parts**

We explicitly point out that spare and accessory parts that have not been supplied by Schütz-Dental have not been tested and released by us. Consequently, the installation and / or use of such products can have a negative influence on the operating characteristics of the machine. The manufacturer cannot be held liable for damage resulting from the use of non-original parts and accessories.

**Note !**

- Spare and wearing parts are always replaced completely. They can not and will not be repaired.
- Return any deficient components to Schütz-Dental.

### **2.4 Switching the device off**

**Caution !**

Adhere to the following switching-off procedure prior to doing any cleaning or replacement of components:

- Set the main switch to OFF
- or disconnect the device from the mains.

Non-compliance will cause danger to the life of your staff.

### 3 Short description

The Schütz-Dental "Welder" was developed to enable everyone to weld dental objects with similar-to-laser quality, but with much less financial engagement.

"Welder" enables you to connect and repair all conventional dental alloys as well as titanium and electro-formed objects.

Schütz-Dental GmbH will not be liable for any use of the system other than the applications described in this manual. Schütz-Dental GmbH will also not be liable for the produced welds. Each user is responsible for checking the quality of their own weldings.

### 4 Technische Daten des Gerätes

#### Dimensions

Width:	390 mm
Height:	450 mm
Depth:	460 mm
Weight:	23 kg

#### Electrical Ratings:

Mains voltage:	230 V~
Frequency:	50/60 Hz
Power consumption	1000 W

#### Disposal of the unit

This unit has been assembled mostly from electronic parts and therefore must be disposed of according to local environmental regulations.

#### Disposal of material residue

These materials must be disposed of in accordance to the manufacturers' instructions and local regulations.

## 5 Transport and packaging

Schütz-Dental-machines and equipment are carefully inspected before they are packed and dispatched. Even so, damage during transport cannot be excluded.

### Incoming inspection:

Check the completeness of the consignment against the delivery note!

### Has the packaging been damaged:

Visually inspect the delivery for damage!

### In the event of complaints:

If the consignment has been damaged during transport:

- Immediately contact the last forwarder. The notification period for packaging damaged during transport by rail is one week; by post 24 hours; by a forwarder 4 days. In the event that the contents are damaged the notification period for transportation by post is 24 hours; by DPD 24 hours and by a forwarder 4 days.
- Please keep the packaging (for possible verification by the forwarder or to return the consignment)!

### Packaging for return transport


If possible use the original package and original packaging material for return transport. If these are no longer available then ensure that the item is packed shock-proof in a cardboard box. Please contact Schütz-Dental if there are any questions concerning the packaging and possible transport lock.

### Storage:

The packaging for the supplied item is not intended for an extended storage period (up to 1 month).

## 6 Set up / Starting the operation

- Remove the packaging.
- Set up the system on an even, non-skid surface.
- Connect the system to a 230 V~ power supply.
- Connect the system to compressed air supply.
- Connect the system to argon gas supply.


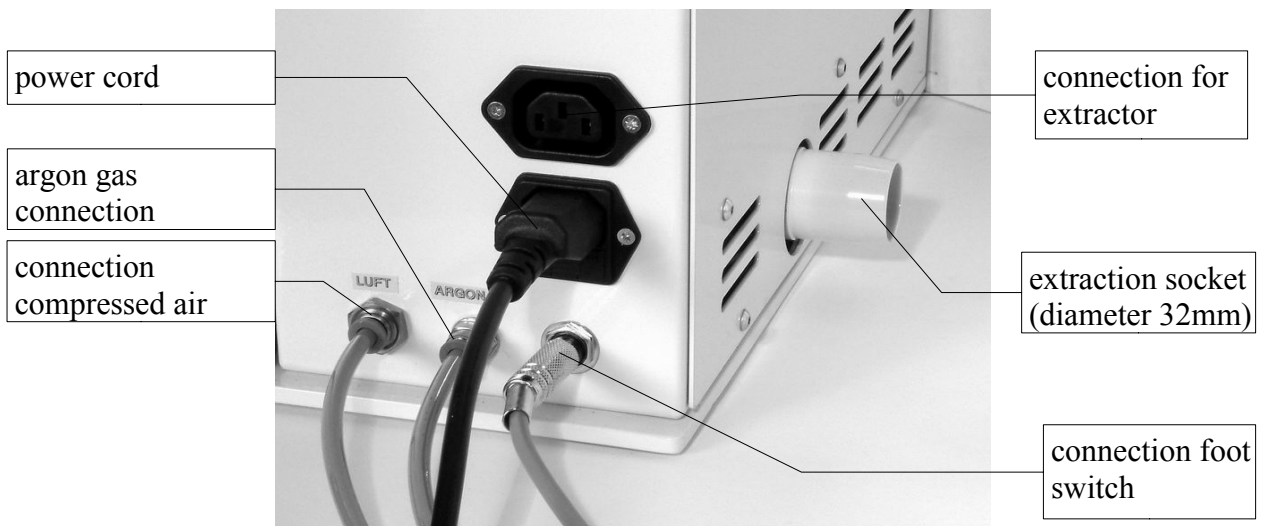


### 6.1 Starting the operation

- Switch on with main switch (acoustic signal, control lamp on).
- Open the main valve of the argon gas container.
- Set the pressure-reducing valve on the argon gas (quality 4.6) container to 4-6 l per minute.
- Press the button "argon" and flood for about 1 sec., if the system has not been in use for some time.

## 7 Operating elements


### 7.1 Operational field

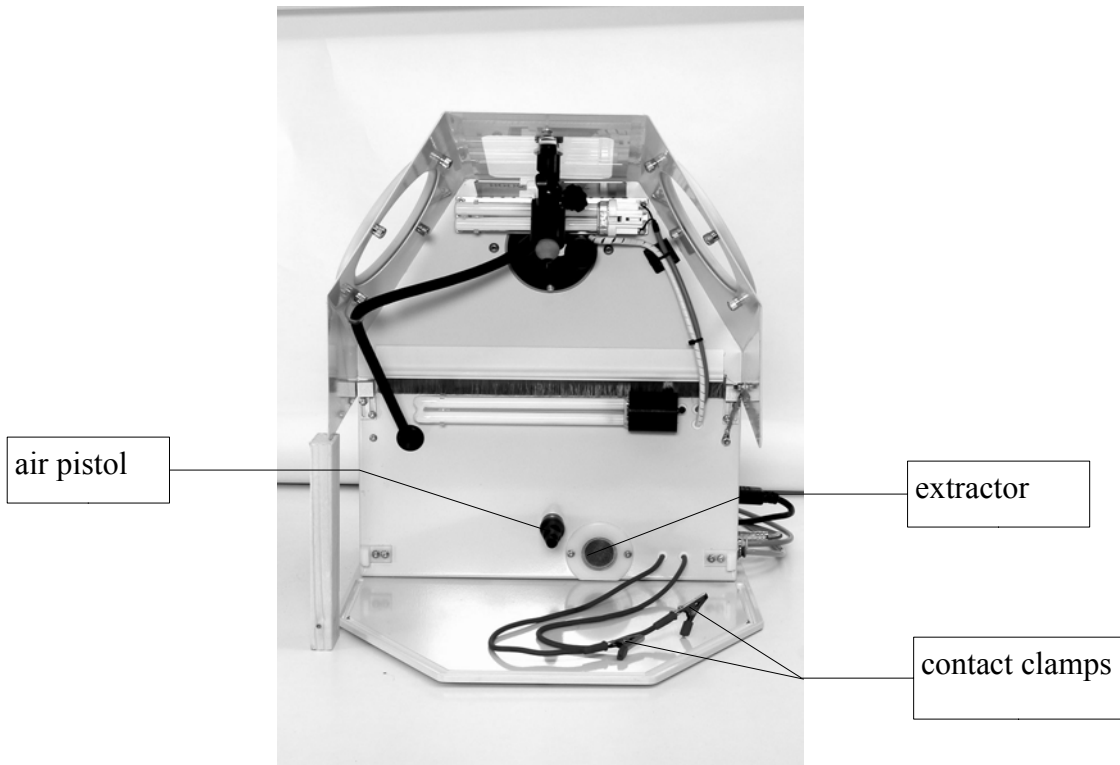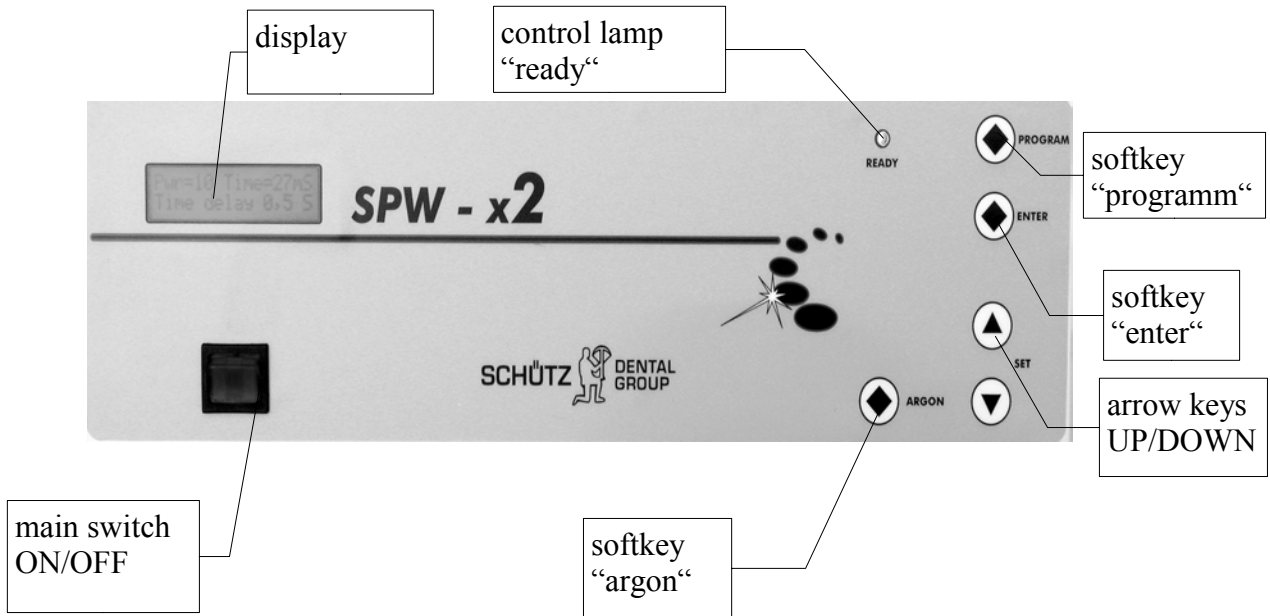

system open

## 8 Operation

- a) For objects, which are in one piece, only one contact clamp is necessary.
- b) One contact clamp should be applied to objects, which are in two pieces (e. g. two separate bridgeworks) - one contact clamp on each piece.

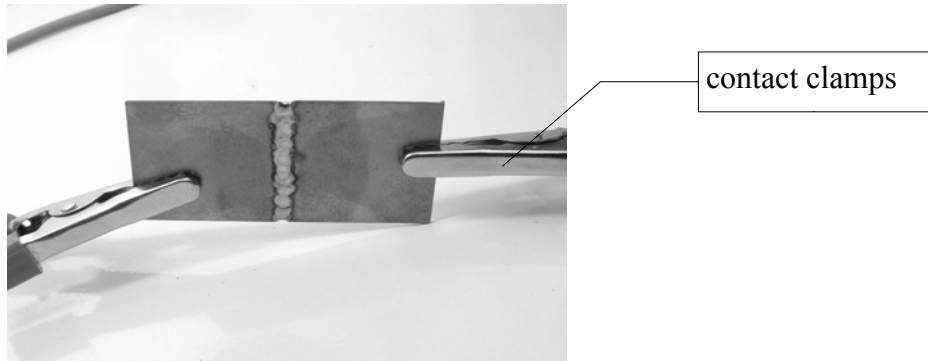

### 8.1 Selecting the programm

Select the desired programm by pressing "programm", select with "set" and confirm with "enter".

- programm 0 = free selection
- programm 1 = basic programm titanium/titanium
- programm 2 = basic programm non-precious alloy/non-precious alloy
- programm 3 = basic programm high-precious alloy/high-precious alloy
- programm 4 = basic programm high-precious alloy/non-precious alloy
- programm 5 = free selection

### 8.2 Changing a programm

Press "programm"

- select desired programm with "set"
- call up each parameter by repeatedly pressing "programm", they can be changed with "set"

### 8.3 Save programmes

- select desired programm with "set"
- call up each parameter by repeatedly pressing "programm", they can be changed with "set"
- press "programm" for 3 sec., an acoustic signal will sound in confirmation

### 8.4 Microscope

Adjust interocular distance and sharpness in order to see the object well.

### 8.5 Sequence of operations

- Set the object into the correct position. The distance to the electrode should be about 1 mm.

**Attention!**

If the electrode should be vitreous, the layer must be removed with a surface planing stone.

- The electrode may not come into contact with the object.
- After positioning the object and, if necessary, position the joining material, start the process with the foot switch (control lamp "ready" turns off).
- After the control lamp relights respectively after the acoustic signal, the next shot can be initiated. If the user keeps pressing the footswitch, follow-up shots are possible.

<b>Programm</b>	<b>Power 1 - 10</b>	<b>Argon 0,5 – 1,5</b>	<b>Timedelay Impulsduration 3ms -99ms</b>
basic programm 0 - freely selectable -			
basic programm 1 titanium/titanium	5	1,0	20 ms
basic programm 2 non-precious alloy / non-precious alloy	7	1,0	25 ms
basic programm 3 high-precious alloy / high-precious alloy	9	1,0	40 ms
basic programm 4 non-precious alloy / high-precious alloy	9	1,0	35 ms
basic programm 5 - freely selectable -			

**Note**

If the objects are very delicate (e. g. clasp arms) respectively very large, power and impulse duration must be changed accordingly. Check the preset programmes and modify them according to your wishes:

- power 1 = lowest output
- power 10 = highest output
- impulse: longer impulse duration = longer flow of energy

## 9 Maintenance / Cleaning

**Danger!!!**

Before doing maintenance or exchanging parts, always turn off the unit as follows:

- switch off with main switch
- pull plug from wallsocket

**Non-observances of these guidelines could cause danger for life and limb.**

### 9.1 Cleaning

The unit is best cleaned with a damp cloth.

**Note !**

Do not use solubles to clean the unit. Solubles could damaged the acrylic parts.

## 10 Emergency

- Switch off unit with main switch.
- Pull plug from wallsocket

## 11 Demounting / disposal

Demounting and disposal are to be carried out in compliance with local environmental guidelines.

## 12 Spare parts

For any questions concerning spare parts, please contact Schütz-Dental at fax +49/60 03/814-906 or phone +49/60 03/814-0.

## **13 Guarantee**

This unit has been constructed and manufactured under consideration of all safety regulations. Should the unit show any manufacturing errors or irregularities during the guarantee period, they will be fixed by the manufacturer according to our standards. For any guarantee claim, the invoice or delivery note must be provided as proof. The guarantee is limited to the replacement of the part or parts which show the manufacturing error or irregularity.

Excluded from guarantee are: personnel costs, expenses of the technical staff, transport costs, packaging, etc. The claim of the following is absolutely ruled out: light bulbs, fuses, glass. Furthermore, errors or other problems which the manufacturer cannot be blamed for or parts which are subject to normal wear may not be claimed under guarantee.

Direct or indirect damage claims towards any persons or things may not be derived from this guarantee, even if the reason for the damage could be traced back to the unit. This guarantee will automatically be terminated, if the unit is repaired, changed or opened by force by the user/buyer or any unauthorized third party.

Should any problem occur during the guarantee period, the user/buyer must only contact the manufacturer or an authorized service center named by the manufacturer.

The right to exchange the complete unit for a new one is excluded. Any parts replaced during the guarantee period must be returned to Schütz-Dental, who will supply a new part. Should the replaced part not be returned, it will be billed to the user/buyer.